

**rodger**

**Amigo**



**Wearable alarm**

# Amigo

The wearable alarm

A design by Rodger

## Wearable alarm

The **Amigo** alarm offers the child an extra helping hand that it needs to get dry. The Amigo is small and light so that it can be carried easily. The Amigo alarm is available in two different variants.

## Treatment of daytime wetting

The **Amigo** alarm has been specially developed for the treatment of daytime wetting. The alarm system can for example be attached to clothing by means of a clip. In case of urine loss, the Amigo gives a vibration and/or sound signal, so that the child knows that he/she has started urinating and has to go to the toilet.

## The two variants of the Amigo alarm are:

- *Amigo Wireless*

The first variant of the **Amigo** comes with a transmitter. The **Wireless Amigo** is used in combination with the **Rodger Sensor Briefs**. This ensures the ultimate comfort. The transmitter is attached to the briefs with press studs. The **Amigo** goes off when urine enters the sensor briefs.

- *Amigo Button*

The second variant comes with the **Rodger Cable** and can be used in combination with the **Rodger Sensor Briefs**. The cable is attached to the press studs of the sensor pants. The **Amigo Button** is easy to operate and has a good connectivity.

## Specifications:

- Weight: 144 grams
- Diameter Amigo: 6 cm
- Cable: 80 cm (without button)
- Button: 4x1,5 cm
- Batteries: 2x AAA
- Input Voltage 2.0 – 2.4V
- Alarm and vibration function

## Helpdesk:

E-mail: [info@rodger.nl](mailto:info@rodger.nl)

Telephone: +31(0)541-351449 on working days from 9 a.m. to 5.00 p.m.

## Contact:

Rodger BV  
Langakker 5c  
5371EV Ravenstein  
The Netherlands

## Warranty:

**The Rodger Amigo** carries a two-year warranty, from receipt of the device by the purchaser, against manufacturer's defects and against breakage or failure under normal use - at our discretion.

This warranty does not cover damage caused by the negligent act or failure to follow the instructions in the manual stated.

This warranty is valid only if you are the original purchaser and have the receipt and proof of purchase. The warranty is not valid if repairs to the device are made by third parties. **Rodger** reserves the right to change specifications and warranty without prior notice.





# Short Manual Amigo Wireless:


## Package content:




- 2x AAA batteries
- 1 x CR2032 battery
- 1 x short manual
- 1 x Rodger Amigo
- 1 x transmitter
- Calendar + sticker sheet
- Small screwdriver
- Warning booklet

# Installation steps:

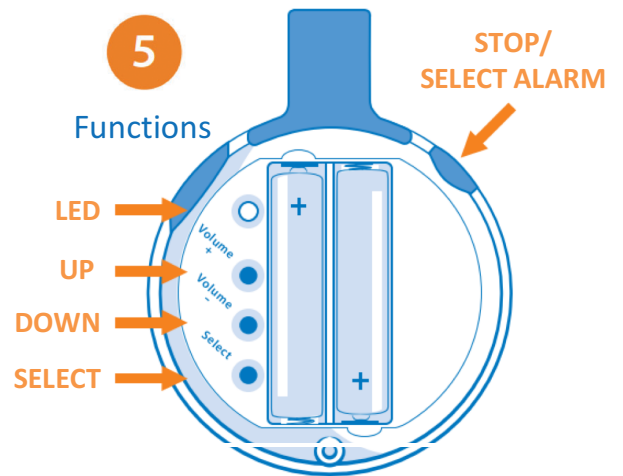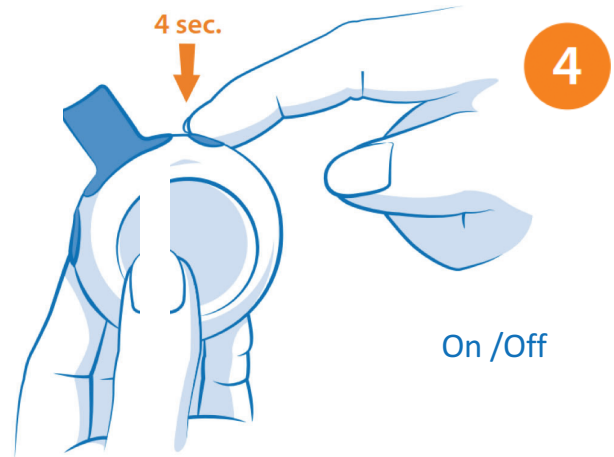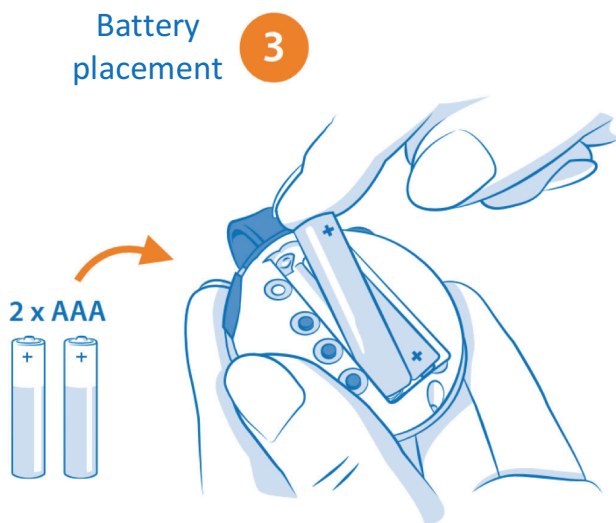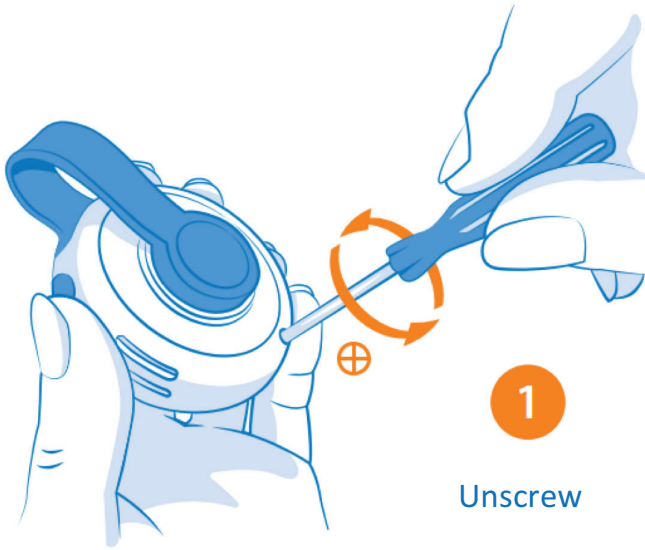


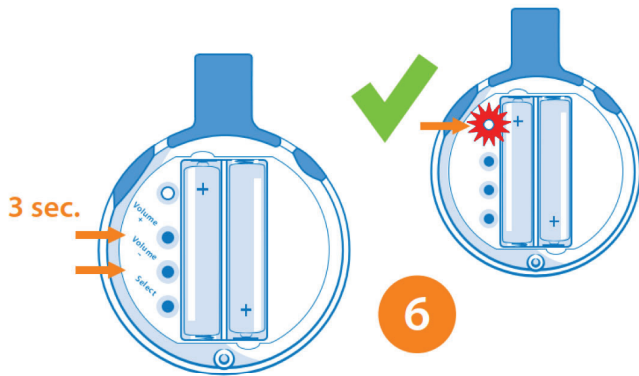

**6**  
Pairing:  
Press + and - simultaneously

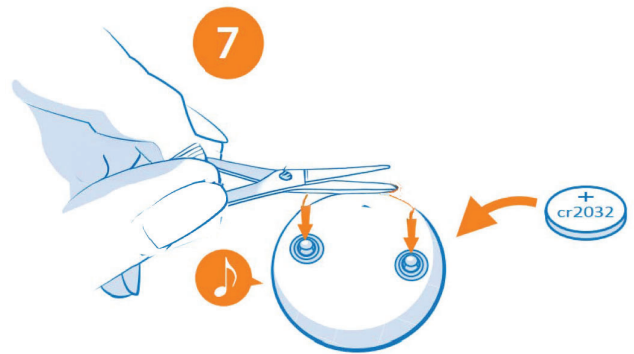


**7**  
Connect contacts with metal  
(example: scissors)

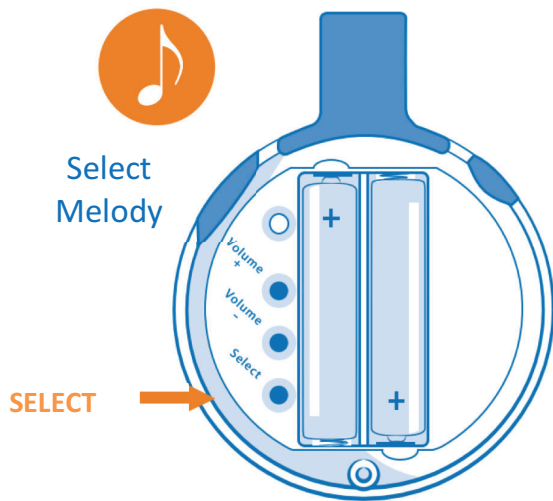

Select  
Melody


SELECT

Volume control

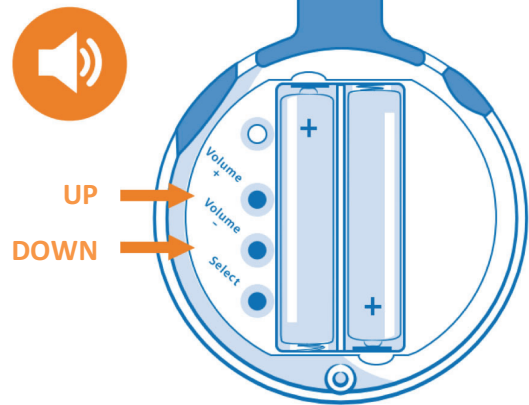

UP

DOWN

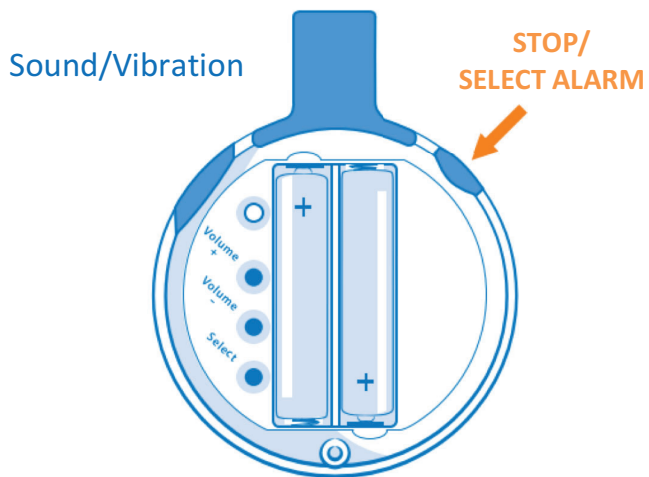

Sound/Vibration

STOP/  
SELECT ALARM

Manufactured under ISO9001.  
Engineered in Holland.  
2 x AAA-batteries included.




# Short Manual Amigo Button:

## Package content:



- 2x AAA batteries
- 1 x short manual
- 1 x Rodger Amigo
- 1 x Rodger Button cable
- Calendar/sticker sheet
- Small screwdriver
- Warning booklet






## Quick overview:

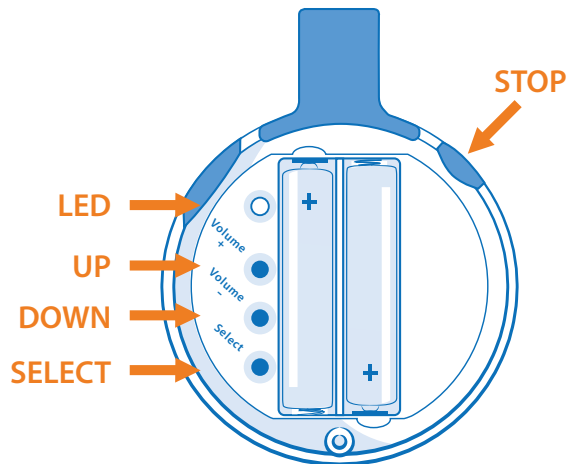

### Buttons and indications

1. Status-LED
2. Volume up button (UP)
3. Volume down button (DOWN)
4. Select/auto button (SEL)
5. Stop/Vibration/Sound button (STOP)


# Installation steps:

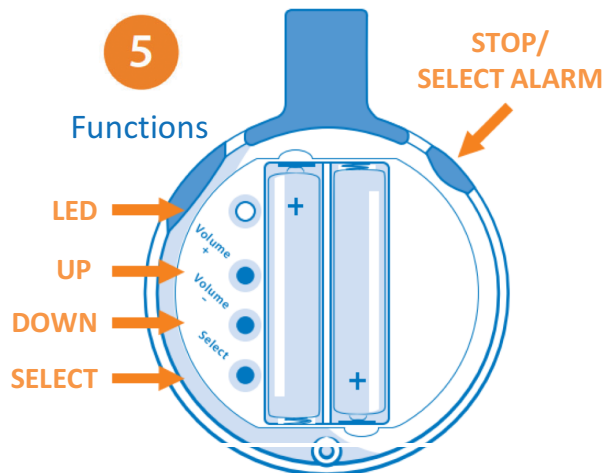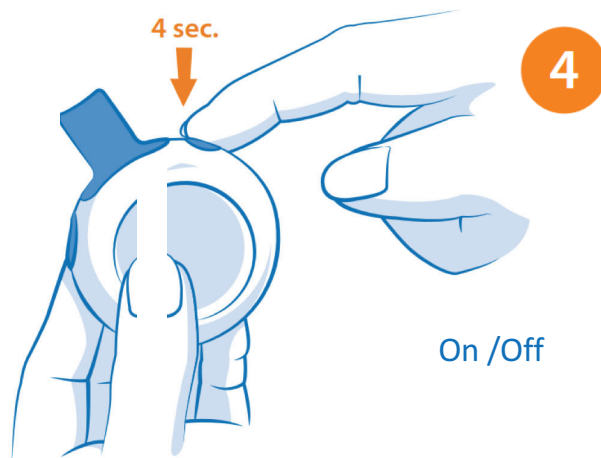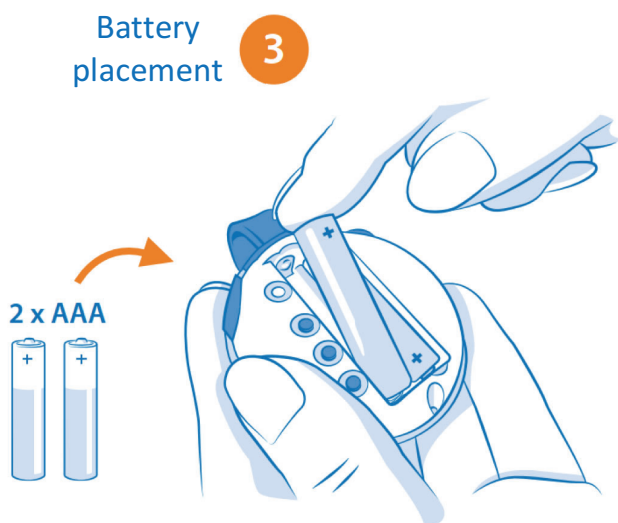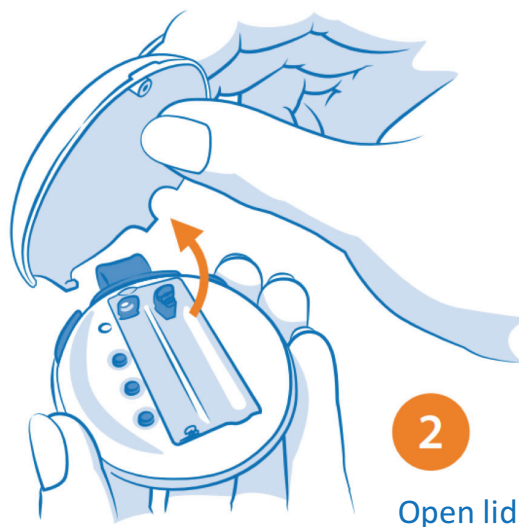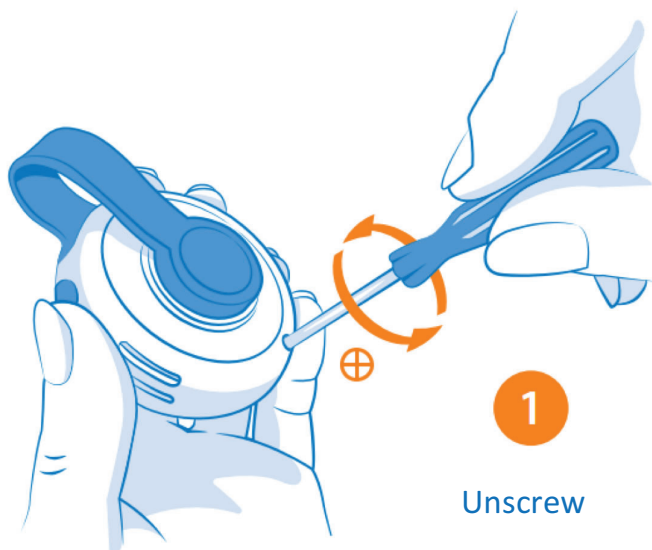

Select Melody




SELECT

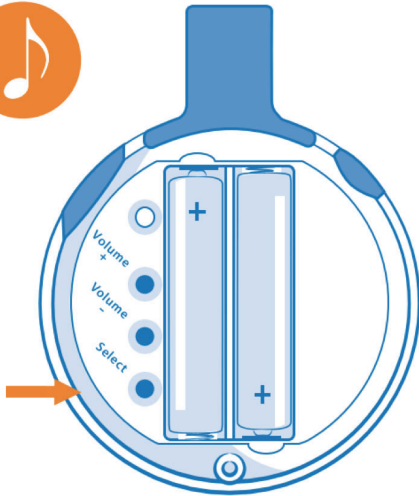

Volume control






UP  
DOWN

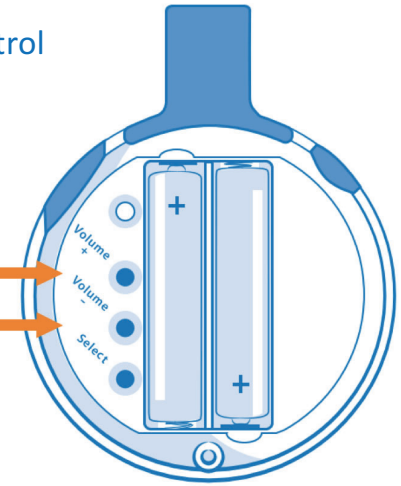

Sound/Vibration

STOP/  
SELECT ALARM

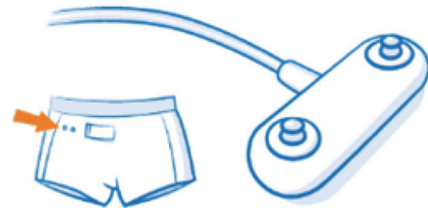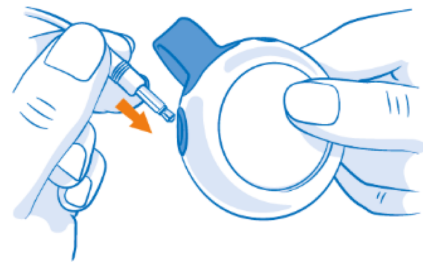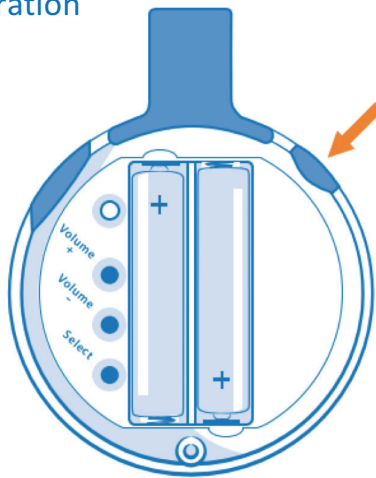

Manufactured under ISO9001.  
Engineered in Holland.  
2 x AAA-batteries included.

